



# Silver Nursing Cups

For use on sore  
breastfeeding nipples



## Care Instructions and Storage

### Material

Our silver nursing cups are **made of 100% pure 999 fine silver**. As silver is a natural product, it can tarnish after a while, if it is not properly stored and cared for. This is a natural process and has no influence on the efficacy of the nursing cups, but rather confirms the purity of the material. However, the aesthetics of the silver cups can be preserved by following the product care instructions.


### Important!

After each use, the silver nursing cups should be cleaned with water and a mild soap. After that, they should be carefully dried with a kitchen towel and stored in the little pouch they are delivered in. Exposing the silver cups to air and humidity increases the risk of discoloration (oxidation).

### Safety and purity

We have our silver nursing cups tested by accredited testing laboratories, including the TÜV Süd. Your product is safe and contains no other metals such as nickel, copper or lead.

**Prüfprotokoll**  
Test record



**Chemische Stückanalyse – Record for Chemical Checkanalysis**  
(Röntgenfluoreszenzanalyse)

Auftrags Nr. – Order No. : 3495530  
 Prüfstück Nr. – Sample No. : Livella Silberhütchen  
 Datum: 25.08.2021  
 Prüfer: M. Becker


Geräte-Nr. – Equipment No. : QS-004 10303

Angaben in Gewichts-Prozent – Requirement in Weight-Percentage (%)

Probe-Nr. Sample- No.	Proben- Bezeichnung Identification	Ag	Fe	Cr	Ni	Mo	Cu	V	Nb	Ti	W	Co	Sn	Pb
1		99.96	0.03	N/A	N/A	N/A	N/A	N/A	N/A	N/A	N/A	N/A	N/A	N/A
2		99.94	0.03	N/A	N/A	N/A	N/A	N/A	N/A	N/A	N/A	N/A	N/A	N/A

Interpretation:  
 Die untersuchten Livella Silberhütchen weisen einen Silberanteil von über 99,9% auf (99,9% ± 0,3%).  
 Interpretation: The analyzed Livella Silver hats have a Silver content above 99.9% (99.9% ± 0.3%).

Prüfung gemäß Methodenbeschreibung QMA-Nr. MUC-WMR-A016:2020



erstellt/gedruckt: S-ANZ-MUC  
 04/2021 M. Becker  
 inhalt/Qualitätsmanagement/QM\_Labor/Prüfstände/  
 597A

Integriert in: Beyer  
 Rev.-Stand: 04/2021

Die Prüfergebnisse beziehen sich ausschließlich  
 auf den untersuchten Prüfgegenstand!  
 Gedruckte Exemplare unterliegen keinem Änderungsprozess!  
 Copyright TÜV SÜD Industrie Service GmbH

Seite 1 von 1





## Care Instructions

### Daily cleaning between uses

We recommend cleaning and rinsing the nursing cups thoroughly with a mild soap and lukewarm water between uses. Afterwards, they should be dried off with a kitchen towel. **Regular rinsing prevents the discolouration of the silver.**

💧 **Cleaning utensils:** water, mild soap, kitchen towel

🕒 **Duration:** approx. 1 minute

### Preventing discolouration after longer periods of use

#### Sodium bicarbonate & water mix

You can clean your Livella nursing cups with a mixture of sodium bicarbonate (baking soda) and water. In a small bowl, combine the baking soda (approx. 2-3 tablespoons) with a small amount of water until you have a slightly liquid mixture. Then apply the mixture to the silver and polish it using a soft, small brush. The tarnish should gradually come off as you brush the silver, leaving it more and more shiny. Finally, clean the silver cups thoroughly with plenty of water and a mild soap before drying them off.

💧 **Cleaning utensils:** sodium bicarbonate (baking soda), brush (e.g. toothbrush) or sponge, bowl, water

🕒 **Duration:** approx. 5 minutes

#### Sodium bicarbonate & salt bath

Start by lining a bowl with a sheet of kitchen foil. Then pour hot (but not boiling) water into the bowl and add 1 tablespoon of sodium bicarbonate and 1 tablespoon of salt. Place the silver nursing cups in the bowl and leave them to soak in the mixture for 1-2 minutes. **Important! Don't leave them in the mixture for more than 2 minutes!** Take the nursing cups out and rinse them thoroughly under running water before drying them off with a kitchen towel.

💧 **Cleaning utensils:** 1 tablespoon of sodium bicarbonate, 1 tablespoon of salt, kitchen foil, bowl, hot water

🕒 **Duration:** approx. 5 minutes



### White toothpaste

Alternatively, the Livella nursing cups can also be polished with a regular white toothpaste. To do this, simply apply some toothpaste to the nursing cups and polish them with a toothbrush. Leave the toothpaste on the cups for approx. 10 minutes before brushing it off with the toothbrush and some water. Finally, rinse the nursing cups thoroughly under running water and dry them off with a cloth.

💧 **Cleaning utensils:** toothpaste, toothbrush, water, kitchen towel

🕒 **Duration:** approx. 15 minutes

### How to store the silver nursing cups

When you don't use your silver nursing cups, you should store them in the little bag in which they are delivered. Exposing them to air and humidity increases the risk of discolouration.

Thank you  
for your trust



We wish you a lovely  
nursing period with your baby.

Your Livella Team